

Pull-it Stent Remover

Description:

Pull-it Stent Remover

Configuration:

See brochure

Intended use:

For removing stents through either rigid or flexible cystoscopy.

Contraindications:

None known

Caution:

Federal (USA) law restricts this device to sale by or on the order of a physician.
Do not pull the stent if there is much resistance to the stent movement.

Warning/ Precaution:

- Do not use if package is opened or damaged.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation
- Storage temperature between 5°C to 30°C.
- Only experienced surgeon should use this device.
- For single use only.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.

Instruction for Use:

The stent remover is advanced through the working channel of the cystoscope with prongs retracted into the sheath. Under vision, the prongs are advanced into the bladder and the end of the stent is grasped tightly giving a slow pull the stent is removed with the whole system.

Do not reuse, reprocess & resterilize:

Reuse, Reprocessing & resterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

Limited Express Warranty:

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages.

The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.



Catalogue
Number



Batch
Code



Date of
Manufacture



Use
By



Do not
re-use



Do not
Re-Sterilize

R_x only

Caution : Federal (USA) law
restricts device to sale by or
on the order of a physician.



Consult
Instruction
for use



Do not use
if Packing
is damaged



Caution



Keep out
of sunlight



Keep
Dry

STERILE EO

Sterilized using
ethylene oxide



30°C
Temperature
Limit



allwin
Medical Devices

Allwin Medical Devices, Inc.

3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA)

Tel. : +1 714-572-1709

E-mail : info@allwinmedical.com | www.allwinmedical.com