

Dilators & Sets

Description:

Dilators & Sets

- ARMADA Ureteral Dilator
- ARMADA Nottingham One Step Dilator
- ARMADA Amplatz Renal Dilator Set
- ARMADA Renal Dilator
- Fascial Dilators

Configuration:

See brochure

Intended use:

- ARMADA Ureteral Dilator: Used for step by step dilatation of the intramural and ureter and / or for stone manipulation.
- ARMADA Nottingham One Step Dilator : One step dilatation for the intramural portion and ureter
- ARMADA Amplatz Renal Dilator Set : Used for progressive dilatation for tract prior to percutaneous kidney stone removal
- ARMADA Renal Dilator: Used for dilatation of Renal Tract
- Fascial Dilators: Used for dilatation of percutaneous track over the guidewire.

Caution:

Federal (USA) law restricts this device to sale by or on the order of a physician.

Dilatation should be performed under fluoroscopic control.

Warning / Precaution:

- Do not use if package is opened or damaged.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- For single use only.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Storage temperature between 5°C to 30°C.

Instruction for Use:















Pass the dilator over the guidewire and gently rotate and push into the track. For Amplatz Dilator over 18F, introducer catheter is advanced over the guidewire which act as guide for the dilators.




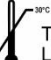

Do not reuse, reprocess & resterilize:

Reuse, Reprocessing & resterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

Limited Express Warranty:

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.

 REF	Catalogue Number	 LOT	Batch Code	 Date of Manufacture	 Use By	 Do not re-use	 Do not Re-Sterilize	 Rx only	Caution: Federal Law restricts this device to sale by or on the order of a physician or a practitioner trained in its use.
 Consult Instruction for use	 Do not use if Packing is damaged	 Caution	 Keep out of sunlight	 Keep Dry	 STERILE	 EO	Sterilized using ethylene oxide		

 EC	 REP	CMC Medical Devices & Drugs S. L. C/Horacio Lengo N° 18, CP 29006, Malaga, Spain Tel.: +34 951 214 054 E-mail : info@cmcmedicaldevices.com	 CE 0123	 Temperature Limit	 allwin Medical Devices Allwin Medical Devices Plot # 223, 230 & 221, Surat Special Economic Zone, Sachin, Surat - 394230 Gujarat. INDIA Tel: +91 99870 84561 E-mail : info@allwinmedical.com www.allwinmedical.com
--	--	---	---	--	--