

Dilators & Sets

Description:

Dilators & Sets

- ARMADA Ureteral Dilator
- ARMADA Nottingham One Step Dilator
- ARMADA Amplatz Renal Dilator Set
- ARMADA Renal Dilator
- Fascial Dilators
- Introducer Sheath
- Introducer Catheter

Configuration:

See brochure

Intended use:

- ARMADA Ureteral Dilator: Used for step by step dilatation of the intramural and ureter and / or for stone manipulation.
- ARMADA Nottingham One Step Dilator : One step dilatation for the intramural portion and ureter
- ARMADA Amplatz Renal Dilator Set : Used for progressive dilatation for tract prior to percutaneous kidney stone removal
- ARMADA Renal Dilator: Used for dilatation of Renal Tract
- Fascial Dilators: Used for dilatation of percutaneous track over the guidewire.
- Introducer Sheath: Used for dilatation of tract over the guidewire.
- Introducer Catheter: Used for dilatation of tract over the guidewire.

Caution:

Federal (USA) law restricts this device to sale by or on the order of a physician.

Dilatation should be performed under fluoroscopic control.

Warning / Precaution:

- Do not use if package is opened or damaged.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- For single use only.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Storage temperature between 5°C to 30°C.

Instruction for Use:

Pass the dilator over the guidewire and gently rotate and push into the track. For Amplatz Dilator over 18F, introducer catheter is advanced over the guidewire which act as guide for the dilators.

Do not reuse, reprocess & resterilize:

Reuse, Reprocessing & resterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

Limited Express Warranty:

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.



Catalogue
Number



Batch
Code



Date of
Manufacture



Use
By



Do not
re-use



Do not
Re-Sterilize

R_x only

Caution: Federal Law restricts this device to sale by or on the order of a physician or a practitioner trained in its use.



Consult
Instruction
for use



Do not use if
Packing is damaged



Caution



Keep out
of sunlight



Keep Dry



Sterilized using
ethylene oxide



CMC Medical Devices & Drugs S. L.
C/Horacio Lengo N° 18, CP 29006, Malaga, Spain
Tel.: +34 951 214 054
E-mail : info@cmcmedicaldevices.com



Allwin Medical Devices, Inc.
3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA)
Tel. : +1 714-572-1709
E-mail : info@allwinmedical.com | www.allwinmedical.com