

## Meatal Dilator - Adult and Pediatric

**Description:**

Meatal Dilator - Adult

Meatal Dilator - Pediatric

**Configuration:**

See brochure

**Intended use:**

Used for dilatation of penile urethral meatus.

**Contraindications:**

None Known

**Caution:**

Federal (USA) law restricts this device to sale by or on the order of a physician.

**Warning/ Precaution:**

- Do not insert with too much pressure.
- Do not use if package is opened or damaged.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- For single use only.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Storage temperature between 5°C to 30°C.

**Instruction for Use:**

Apply lubricant jelly into the penile meatus. Slowly insert the dilator into meatus for dilatation.

**Do not reuse, reprocess & resterilize:**

Reuse, Reprocessing & resterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

**Limited Express Warranty:**

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.



Catalogue  
Number



Batch  
Code



Date of  
Manufacture



Use  
By



Do not  
re - use



Do not  
Re-Sterilize

**R<sub>x</sub>** only

Caution : Federal (USA) law  
restricts device to sale by or  
on the order of a physician.



Consult  
Instruction  
for use



Do not use  
if Packing  
is damaged



Caution



Keep out  
of sunlight



Keep  
Dry

STERILE EO

Sterilized using  
ethylene oxide



30°C  
Temperature  
Limit



**allwin**  
Medical Devices

**Allwin Medical Devices, Inc.**

3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA)

Tel. : +1 714-572-1709

E-mail : [info@allwinmedical.com](mailto:info@allwinmedical.com) | [www.allwinmedical.com](http://www.allwinmedical.com)