

allFlex Ureteral Catheter

Description:

- Ureteral Catheter (Open Tip)
- Ureteral Catheter (Open Tapered Tip)
- Ureteral Catheter (Closed Tip)
- Ureteral Catheter (Cone Tip)
- Ureteral Catheter (Angled Open Tip)
- Ureteral Catheter (Angled Closed Tip)

Configuration:

See brochure

Intended use:

The Allwin Ureteral Catheters are used for drainage of urine from the ureter and they allow the delivery of contrast media. The target population is for adults only (greater than 21 years of age)

Contraindications:

None known

Caution:

Federal (USA) law restricts this device to sale by or on the order of a physician.

Warning/ Precaution:

- Do not withdraw ureteral catheter while it is deflected in endoscope.
- Avoid sharp bending.
- This unit should not be used as ureteral splint.
- For single use only.
- Attempt to sterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Do not use if package is opened or damaged.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- Only experienced surgeon should place this device.
- Storage temperature between 5°C to 30°C.

Instruction for Use:

Ureteral Catheters are designed in a variety of distal tip configurations. The ureteral catheter should be inserted into the ureteral through a urological endoscope. Check the deflection on the endoscope for sharp and rough edges before using ureteral catheter. Ureteral catheter adapter with luer lock is used for irrigation.

Removal of Catheter:

Catheter removal should be under fluoroscopic guidance, to ensure safe removal.

Do not reuse, reprocess & resterilize:

Reuse, Reprocessing & resterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

Limited Express Warranty:

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.



Catalogue
Number



Batch
Code



Date of
Manufacture



Use
By



Do not
re-use



Do not
Re-Sterilize

R_x only

Caution: Federal Law restricts this device to sale by or on the order of a physician or a practitioner trained in its use.



Consult
Instruction
for use



Do not use if
Packing is damaged



Caution



Keep out
of sunlight



Keep Dry



STERILE

Sterilized using
ethylene oxide



CMC Medical Devices & Drugs S. L.
C/Horacio Lengua N° 18, CP 29006, Malaga, Spain
Tel.: +34 951 214 054
E-mail : info@cmcmedicaldevices.com



allwin
Medical Devices

Allwin Medical Devices, Inc.
3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA)
Tel. : +1 714-572-1709
E-mail : info@allwinmedical.com | www.allwinmedical.com