

## U-Perc Nitinol Stone Basket

**Description:**

U-Perc Nitinol Stone Basket

**Configuration:**

See brochure

**Intended use:**

U-Perc Nitinol Stone Basket is used for stone extraction during percutaneous nephrolithotomy (PCNL.)

**Contraindications:**

Not suited for use as a lithotripter basket.

**Complications:**

Some possible complications may include, but are not limited to, the following:

- Perforation
- Evulsion
- Edema
- Entrapment
- Hemorrhage
- Inability to disengage from irretrievable object

**Caution:**

Federal (USA) law restricts this device to sale by or on the order of a physician.

**Warning/ Precaution:**

- Manipulate the handle prior to use to ensure smooth functioning of the device. Avoid contact with electric instruments.
- Only physicians trained in stone manipulation should perform this procedure. A variety of techniques can be employed; however the physician should employ the technique with which he/she is most comfortable.
- For single use only.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Do not use if package is opened or damaged.
- Check compatibility with working channel of endoscope.
- Storage temperature between 5°C to 30°C.

**Potential Biohazard:**

Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.

**Instruction for Use:**

- Insert U-Perc through the working channel of the endoscope for capturing calculi.
- Once the calculus is entrapped in the basket, gradual pull is requested so that there is no mucosal damage.
- Fluoroscopic visualization is advantageous.
- If unsuccessful, further attempts may be made by repeating previously described procedure.
- Simultaneously, withdraw the instrument and the endoscopes from the area of interest.

**Do not reuse, reprocess & resterilize:**

Reuse, Reprocessing & reesterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

**Limited Express Warranty:**

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.

 Catalogue Number  
  Batch Code  
  Date of Manufacture  
  Use By  
  Do not re-use  
  Do not Re-Sterilize  
 Rx only  
 Caution: Federal Law restricts this device to sale by or on the order of a physician or a practitioner trained in its use.

 Consult Instruction for use  
  Do not use if Packing is damaged  
  Caution  
  Keep out of sunlight  
  Keep Dry  
  STERILE EO  
 Sterilized using ethylene oxide

 CMC Medical Devices & Drugs S. L.  
 C/Horacio Lengua N° 18, CP 29006, Malaga, Spain  
 Tel.: +34 951 214 054  
 E-mail : info@cmcmedicaldevices.com

 CE 0123  
  30°C Temperature Limit


  
**Allwin Medical Devices**  
 Plot # 223, 230 & 221, Surat Special Economic Zone, Sachin,  
 Surat - 394230 Gujarat. INDIA | Tel: +91 99870 84561  
 E-mail : info@allwinmedical.com | www.allwinmedical.com