

GRASP-IT Three – Prong Grasping Forcep

Description:

GRASP-IT Three – Prong Grasping Forcep

Configuration:

See brochure

Intended use:

The GRASP-IT Three Prong Grasping Forcep is designed for endoscopically manipulating, dislodging, engaging and removing renal and ureteral stones.

Contraindications:

No known contraindications.

Complications:

Some possible complications may include, but are not limited to, the following:

- Perforation
- Evulsion
- Edema
- Entrapment
- Hemorrhage
- Inability to disengage from irretrievable object

Caution:

Federal (USA) law restricts this device to sale by or on the order of a physician.

Warning/ Precaution:

- Manipulate the handle prior to use to ensure smooth functioning of the device. Avoid contact with electric instruments.
- Check compatibility with working channel of endoscope.
- Only physicians trained in stone manipulation should perform this procedure. A variety of techniques can be employed; however the physician should employ the technique with which he/she is most comfortable.
- For single use only.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Do not use if package is opened or damaged.
- Storage temperature between 5°C to 30°C.

Potential Biohazard:

Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.

Instruction for Use:

- Retract the forceps into the catheter shaft.
- Insert the catheter shaft into the working channel of the endoscopes.
- With the tip still retracted, advance the instrument to the stone to be removed which is fluoroscopically / endoscopically verified.
- Manipulate the handle to open the instrument slowly. Once the stone has been captured, partially close the instrument to secure the stone for removal.
- If unsuccessful, further attempts may be made by repeating previously described procedure.
- Simultaneously, withdraw the instrument and the endoscopes from the area of interest.

Do not reuse, reprocess & resterilize:

Reuse, Reprocessing & resterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

Limited Express Warranty:

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages.

The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.



Catalogue
Number



Batch
Code



Date of
Manufacture



Use
By



Do not
re-use



Do not
Re-Sterilize



Rx only

Caution: Federal Law restricts this device to sale by or on the order of a physician or a practitioner trained in its use.



Consult
Instruction
for use



Do not use if
Packing is damaged



Caution



Keep out
of sunlight



Keep Dry



STERILE

EO

Sterilized using
ethylene oxide



CMC Medical Devices & Drugs S. L.
C/Horacio Lengo N° 18, CP 29006, Malaga, Spain
Tel.: +34 951 214 054
E-mail : info@cmcmedicaldevices.com



Temperature
Limit



Allwin Medical Devices, Inc.
3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA)
Tel. : +1 714-572-1709
E-mail : info@allwinmedical.com | www.allwinmedical.com