

# MACHO Multiwire Stone Extractor

**Description:**

MACHO Multiwire Stone Extractor

**Intended use:**

Macho multiwire stone extractor is used for endoscopic removal of stones and foreign bodies

**Warning/ Precaution:**

- To be used by thoroughly trained physicians only.
- For single use only.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- Do not use if package is opened or damaged.
- Store in cool and dry place.

**Instruction for Use:**

1. The basket is passed through the endoscope, beyond the stone, before it is fully opened taking care that the stone is not pushed further.
2. It may be necessary to jiggle the basket to trap a stone within it. The basket, then pulled down the duct fully or partially open, may extract the stone.
3. Drag the basket till it meets resistance and when the basket and stone reach the sphincterotomy orifice, advance the endoscope tip onto the papilla.
4. The final stage of extraction is best done using a 'flip-down' maneuver. Holding the basket positioned steady in the papilla, the endoscope tip is angled sharply down and rotated to the right; this exerts force in the correct biliary axis and is usually successful in extracting the stone.


**Do not reuse:**

Reuse may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

**Limited Express Warranty:**

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages.

The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.

 Catalogue Number	 Batch Code	 Date of Manufacture	 Use By	 Do not re-use	 Do not Re-Sterilize	 Rx only	Caution : Federal (USA) law restricts device to sale by or on the order of a physician.
 Consult Instruction for use	 Do not use if Packing is damaged	 Caution	 Keep out of sunlight	 Keep Dry	 STERILE EO	 Sterilized using ethylene oxide	



**Allwin Medical Devices, Inc.**  
 3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA)  
 Tel. : +1 714-572-1709  
 E-mail : info@allwinmedical.com | www.allwinmedical.com