

## Initial Puncture Needles

**Description:**

- Initial Puncture Needle - 3 Part Bevel Tip
- Initial Puncture Needle - 2 Part Trocar Tip

**Configuration:**

See brochure

**Intended use:**

Used for percutaneous nephrostomy and all other percutaneous application where a 0.035" Guidewire is to be used.

**Contraindications:**

No known contraindications.

**Caution:**

Federal (USA) law restricts this device to sale by or on the order of a physician.

**Warning/ Precaution:**

- Single use only.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Do not use if package is opened or damaged.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- Only experienced surgeons should use this device.
- Storage temperature between 5°C to 30°C.

**Instruction for Use:**








Using fluoroscopy / ultrasound advance the needle through cut fascia which is anesthetized previously till it reaches the point of interest. Further steps are done depending on the requirement of the physician.

**Do not reuse, reprocess & resterilize:**


Reuse, Reprocessing & reesterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.


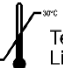
**Limited Express Warranty:**

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.

 Catalogue Number
  Batch Code
  Date of Manufacture
  Use By
  Do not re-use
  Do not Re-Sterilize
  Rx only
 Caution: Federal Law restricts this device to sale by or on the order of a physician or a practitioner trained in its use.

 Consult Instruction for use
  Do not use if Packing is damaged
  Caution
  Keep out of sunlight
  Keep Dry
  STERILE EO
 Sterilized using ethylene oxide


**CMC Medical Devices & Drugs S. L.**  
 C/Horacio Lengo N° 18, CP 29006, Malaga, Spain  
 Tel.: +34 951 214 054  
 E-mail : info@cmcmedicaldevices.com



 Temperature Limit


**Allwin Medical Devices, Inc.**  
 3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA)  
 Tel. : +1 714-572-1709  
 E-mail : info@allwinmedical.com | www.allwinmedical.com