

Chiba Needle - 2 Part Bevel Tip

Description:

Chiba Needle - 2 Part Bevel Tip

Configuration:

See brochure

Intended use:

Chiba Needle is used for percutaneous cholangiography, cytological biopsy and routine percutaneous procedure.

Contraindications:

No known contraindication.

Caution:

Federal (USA) law restricts this device to sale by or on the order of a physician.

Warning / Precaution:

- Single use only
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Do not use if package is opened or damaged.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- Only experienced surgeon should use this device.
- Storage temperature between 5°C to 30°C.

Instruction for Use:

Using fluoroscopy / ultrasound advance the needle through cut fascia which is anesthetized previously till it reaches the point of interest. Further steps are done depending on the requirement of the physician.

Do not reuse, reprocess & resterilize:

Reuse, Reprocessing & resterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

Limited Express Warranty:

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.



Catalogue
Number



Batch
Code



Date of
Manufacture



Use
By



Do not
re-use



Do not
Re-Sterilize

R_x only

Caution: Federal Law restricts this device to sale by or on the order of a physician or a practitioner trained in its use.



Consult
Instruction
for use



Do not use if
Packing is damaged



Caution



Keep out
of sunlight



Keep Dry

STERILE EO

Sterilized using
ethylene oxide



CMC Medical Devices & Drugs S. L.
C/Horacio Lengo N° 18, CP 29006, Malaga, Spain
Tel.: +34 951 214 054
E-mail : info@cmcmedicaldevices.com



Temperature
Limit



Allwin Medical Devices
Plot # 223, 230 & 221, Surat Special Economic Zone, Sachin,
Surat - 394230 Gujarat, INDIA | Tel: +91 99870 84561
E-mail : info@allwinmedical.com | www.allwinmedical.com