

Trackfinder Bulb Irrigator

Description:

Trackfinder Bulb Irrigator

Configuration:

See brochure

Intended use:

Trackfinder bulb irrigator is used for giving continuous and bolus irrigation during endoscopic surgeries.

Contraindications:

None Known

Caution:

Federal (USA) law restricts this device to sale by or on the order of a physician.

Warning / Precaution:

- For Single use only.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Do not use if package is opened or damaged.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- Storage temperature between 5°C to 30°C.
- Only experienced surgeon should use this device.

Instruction for Use:

1. Spike the irrigation solution bottle containing sterile solution by sterile object.
2. Adjust the flow of irrigation solution as required.
3. Connect the luer lock adapter to endoscope irrigation port.
4. Squeeze the bulb for bolus irrigation as and when required.
5. The bulb will automatically refill itself.








Do not reuse, reprocess & resterilize:









Reuse, Reprocessing & reesterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

Limited Express Warranty:

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages.

The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.

 Catalogue Number
  Batch Code
  Date of Manufacture
  Use By
  Do not re-use
  Do not Re-Sterilize
  Rx only
 Caution: Federal Law restricts this device to sale by or on the order of a physician or a practitioner trained in its use.

 Consult Instruction for use
  Do not use if Packing is damaged
  Caution
  Keep out of sunlight
  Keep Dry
  STERILE
  EO
 Sterilized using ethylene oxide
  LATEX
 Not made with natural rubber latex

 
CMC Medical Devices & Drugs S. L.
 C/Horacio Lengo N° 18, CP 29006, Malaga, Spain
 Tel.: +34 951 214 054
 E-mail : info@cmcmedicaldevices.com


 0123


Allwin Medical Devices, Inc.
 3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA)
 Tel. : +1 714-572-1709
 E-mail : info@allwinmedical.com | www.allwinmedical.com