

## Ellick Bladder Evacuator (Glass)

**Description:**

Ellick bladder Evacuator (Glass)

**Configuration:**

See brochure

**Intended use:**

For bladder wash during transurethral resection of prostate (TURP).

**Contraindications:**

None known.

**Caution:**

Federal (USA) law restricts this device to sale by or on the order of a physician.

**Warning / Precaution:**

- Check compatibility of adapter with endoscopes and ensure tight fittings.
- Check the bulb to fit tightly on the plastic reservoir.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- Only experienced surgeons should use this device.
- Storage temperature between 5°C to 30°C.

**Instruction for Use:**

1. Fill the reservoir with irrigation solution.
2. Connect the adapter to the sheath of the resectoscope, press firmly inward.
3. Fill the bladder evacuator with the appropriate volume of irrigation solution.
4. Alternatively, squeeze the bulb multiple times, to remove the tissue from the bladder. Repeat if required.

**Limited Express Warranty:**

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.



Catalogue  
Number



Batch  
Code



Date of  
Manufacture



Use  
By



Do not use if  
Packing is damaged

R<sub>x</sub> only

Caution: Federal Law restricts this device to sale by or on the order of a physician or a practitioner trained in its use.



Consult  
Instruction  
for use



Caution



Keep out  
of sunlight



Keep  
Dry



Temperature  
Limit

CE  
0123



CMC Medical Devices & Drugs S. L.  
C/Horacio Lengo N° 18, CP 29006, Malaga, Spain  
Tel.: +34 951 214 054  
E-mail : info@cmcmedicaldevices.com



**allwin**<sup>®</sup>  
Medical Devices

**Allwin Medical Devices, Inc.**  
3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA)  
Tel. : +1 714-572-1709  
E-mail : info@allwinmedical.com | www.allwinmedical.com