

Ellick Bladder Evacuator (Plastic)

Description:

Ellick bladder Evacuator (Plastic)

Configuration:

See brochure

Intended use:

For bladder wash during transurethral resection of prostate (TURP).

Contraindications:

None known.

Caution:

Federal (USA) law restricts this device to sale by or on the order of a physician.

Warning / Precaution:

- Check compatibility of adapter with endoscopes and ensure tight fittings.
- Check the bulb to fit tightly on the plastic reservoir.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- Only experienced surgeons should use this device.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Storage temperature between 5°C to 30°C.

Instruction for Use:

1. Fill the reservoir with irrigation solution.
2. Connect the adapter to the sheath of the resectoscope, press firmly inward.
3. Fill the bladder evacuator with the appropriate volume of irrigation solution.
4. Alternatively, squeeze the bulb multiple times, to remove the tissue from the bladder. Repeat if required.

Do not reuse, reprocess & resterilize:

Reuse, Reprocessing & resterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

Limited Express Warranty:

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.



Read
Instruction



For Single
Use Only



Do not
Re-Sterilize



Use
By



Sterile unless package
is opened or damaged



Caution : Federal (USA) law restricts device
to sale by or on the order of a physician



Sterilized using
ethylene oxide




Keep out
of sunlight



Keep
Dry



Not made
with natural
rubber latex

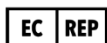

CMC Medical Devices & Drugs S. L.
C/Horacio Lengo N° 18, CP 29006, Malaga, Spain
Tel.: +34 951 214 054
E-mail : info@cmcmedicaldevices.com



allwin[®]
Medical Devices

Allwin Medical Devices, Inc.
3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA)
Tel. : +1 714-572-1709
E-mail : info@allwinmedical.com | www.allwinmedical.com