

## Ellick Bladder Evacuator (Plastic)

**Description:**

Ellick bladder Evacuator (Plastic)

**Configuration:**

See brochure

**Intended use:**

For bladder wash during transurethral resection of prostate (TURP).

**Contraindications:**

None known.

**Caution:**

Federal (USA) law restricts this device to sale by or on the order of a physician.

**Warning / Precaution:**

- Check compatibility of adapter with endoscopes and ensure tight fittings.
- Check the bulb to fit tightly on the plastic reservoir.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- Only experienced surgeons should use this device.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Storage temperature between 5°C to 30°C.

**Instruction for Use:**

1. Fill the reservoir with irrigation solution.
2. Connect the adapter to the sheath of the resectoscope, press firmly inward.
3. Fill the bladder evacuator with the appropriate volume of irrigation solution.
4. Alternatively, squeeze the bulb multiple times, to remove the tissue from the bladder. Repeat if required.

**Do not reuse, reprocess & resterilize:**

Reuse, Reprocessing & resterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

**Limited Express Warranty:**

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.



Read  
Instruction



For Single  
Use Only



Do not  
Re-Sterilize



Use  
By



Sterile unless package  
is opened or damaged



Caution : Federal (USA) law restricts device  
to sale by or on the order of a physician



Sterilized  
using ethylene  
oxide



Keep out  
of sunlight



Keep  
Dry



Not made  
with natural  
rubber latex



**CMC Medical Devices & Drugs S. L.**  
C/Horacio Lengo N° 18, CP 29006, Malaga, Spain  
Tel.: +34 951 214 054  
E-mail : info@cmcmedicaldevices.com



**Allwin Medical Devices**

Plot # 223, 230 & 221, Surat Special Economic Zone, Sachin,  
Surat - 394230 Gujarat. INDIA | Tel: + 91 99870 84561  
E-mail : info@allwinmedical.com | www.allwinmedical.com