

TUR-VAC Bladder Evacuator

Description:

TUR-VAC Bladder Evacuator

Configuration:

See brochure

Intended use:

Used for evacuating irrigation solutions, blood clots, tissue collection and/or training functions during prostate surgery or bladder surgery.

Contraindications:

None Known

Caution:

Federal (USA) law restricts this device to sale by or on the order of a physician.

Warning / Precaution:

- Check compatibility of adapter with endoscopes and ensure tight fittings.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- For single use only.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Only experienced surgeons should use this device.
- Do not use if package is opened or damaged.
- Storage temperature between 5°C to 30°C.

Instruction for Use:




1. Fill the barrel with irrigation solution.
2. Connect the device to the sheath of the resectoscope, press firmly inward.
3. Fill the bladder evacuator with the appropriate volume of irrigation solution.
4. Alternatively, squeeze the barrel multiple times, to remove the tissues from the bladder. Repeat if required




Do not reuse, reprocess & resterilize:



Reuse, Reprocessing & resterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

Limited Express Warranty:

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.

 Catalogue Number
  Batch Code
  Date of Manufacture
  Use By
  Do not re-use
  Do not Re-Sterilize
  Rx only
 Caution: Federal Law restricts this device to sale by or on the order of a physician or a practitioner trained in its use.

 Consult Instruction for use
  Do not use if Packing is damaged
  Caution
  Keep out of sunlight
  Keep Dry
  STERILE EO
 Sterilized using ethylene oxide
  LATEX
 Not made with natural rubber latex

  CMC Medical Devices & Drugs S. L.
C/Horacio Lengo N° 18, CP 29006, Malaga, Spain
Tel.: +34 951 214 054
E-mail : info@cmcmedicaldevices.com

CE
0123


Allwin Medical Devices, Inc.
 3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA)
 Tel. : +1 714-572-1709
 E-mail : info@allwinmedical.com | www.allwinmedical.com