

Toomey Syringe

Description:

Toomey Syringe

Configuration:

See brochure

Intended use:

Toomey Syringe is used for removing debris during endosurgical resection and clots from the bladder.

Contraindications:

No known contraindication

Caution:

- Federal (USA) law restricts this device to sale by or on the order of a physician.
- Secure tip tightly prior to use.
- Avoid excessive syringe pressure when using luer adapter as adapter may dislodge.

Warning / Precaution:

- For single use only
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Do not use if package is opened or damaged.
- After use, this product may be a potential biohazard. Handle and dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- Storage temperature between 5°C to 30°C.
- Only experienced surgeon should use this device.

Instruction for Use:

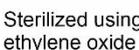
1. Connect the syringe to the sheath and ensure tight fit.
2. Advance and retract the plunger for remove debris.

Do not reuse, reprocess & resterilize:

Reuse, Reprocessing & resterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

Limited Express Warranty:

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.

 Catalogue Number	 Batch Code	 Date of Manufacture	 Use By	 Do not re-use	 Do not Re-Sterilize	 Rx only	Caution : Federal (USA) law restricts device to sale by or on the order of a physician.
 Consult Instruction for use	 Do not use if Packing is damaged	 Caution	 Keep out of sunlight	 Keep Dry	 STERILE EO	 Sterilized using ethylene oxide	 Temperature Limit



Allwin Medical Devices
 Plot # 223, 230 & 221, Surat Special Economic Zone, Sachin,
 Surat - 394230 Gujarat, INDIA | Tel: +91 99870 84561
 E-mail : info@allwinmedical.com | www.allwinmedical.com