

SCN- Sclerotherapy Needle

Description:

SCN- Sclerotherapy Needle

Configuration:

See brochure

Intended use:

Used for injection in to and around bleeding vessels including varices.

Contraindications:

- All contraindications specific to primary endoscopic procedures to be performed in gaining access to desired injection site
- Severe Coagulopathy.

Complications:

- Bleeding
- Perforation
- Pleural effusions
- Hepatic failure
- Post injection ulceration with delayed bleeding.
- Chest Pain
- Esophageal stricture
- Aspiration pneumonia
- Dysphasia
- Septicemia
- Esophageal Ulcers
- Other respiratory difficulties

Caution:

Federal (USA) law restricts this device to sale by or on the order of a physician.

Warning/ Precaution:

- To be used by thoroughly trained physician only.
- Do not use if package is opened or damaged.
- Check the endoscopes working channel diameter compatibility with needle.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
- For single use only.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Storage temperature between 5°C to 30°C.

Instruction for Use:

1. Open package and inspect the needle for defects. Do not use if found defective.
2. Withdraw the needle and ensure that it is totally retracted into the outer catheter.
3. Advance the needle in straight position, through the working channel of the endoscope while the needle tip is embedded in the outer catheter.














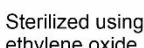

4. Once at the intended site, advance the needle tip by pushing the lower hub, with syringe attached.
5. After injection retract the needle tip completely into the outer catheter and remove the injection needle from the endoscopes working channel.

Do not reuse, reprocess & resterilize:

Reuse, Reprocessing & reesterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

Limited Express Warranty:

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.

 Catalogue Number	 Batch Code	 Date of Manufacture	 Use By	 Do not re-use	 Do not Re-Sterilize	 Rx only	Caution : Federal (USA) law restricts device to sale by or on the order of a physician.
 Consult Instruction for use	 Do not use if Packing is damaged	 Caution	 Keep out of sunlight	 Keep Dry	 STERILE EO	 Sterilized using ethylene oxide	 30°C Temperature Limit



Allwin Medical Devices
 Plot # 223, 230 & 221, Surat Special Economic Zone, Sachin,
 Surat - 394230 Gujarat. INDIA | Tel: +91 99870 84561
 E-mail : info@allwinmedical.com | www.allwinmedical.com