

## CONQUER Retrieval Snare

**Description:**  
CONQUER Retrieval Snare

**Configuration:**  
See brochure

**Intended use:**  
Non-diathermic retrieval snare is used to retrieve foreign objects from the upper G.I. tract.

**Contraindications:**  
There are no known contraindications.

- Complications:**
- Perforation
  - Hemorrhage
  - Mucosal laceration
  - Infection and aspiration














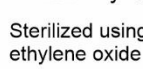

**Caution:**  
Federal (USA) law restricts this device to sale by or on the order of a physician.

- Warning / Precaution:**
- Do not use if package is opened or damaged.
  - Only experienced surgeons should use this device.
  - Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.
  - Product supplied sterile intended for single use only.
  - Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
  - Storage temperature between 5°C to 30°C.
  - Only experienced surgeon should use this device.

**Instruction for Use:**  
An endoscope is introduced and advanced till the object to be removed is seen. The instrument is advanced through the endoscope and the snare / basket opened. Secure the object in the snare / basket; slowly retrieve together with the endoscope. Use of overtube for foreign body removal should be avoided, considering that overtube insertion can be uncomfortable to the patient and may injure esophagus.

**Do not reuse, reprocess & resterilize:**  
Reuse, Reprocessing & resterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

**Limited Express Warranty:**  
The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages. The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.

 Catalogue Number	 Batch Code	 Date of Manufacture	 Use By	 Do not re-use	 Do not Re-Sterilize	 <b>Rx only</b>	Caution : Federal (USA) law restricts device to sale by or on the order of a physician.
 Consult Instruction for use	 Do not use if Packing is damaged	 Caution	 Keep out of sunlight	 Keep Dry	 STERILE EO	 Sterilized using ethylene oxide	 5°C - 30°C Temperature Limit



**Allwin Medical Devices, Inc.**  
3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA)  
Tel. : +1 714-572-1709  
E-mail : info@allwinmedical.com | www.allwinmedical.com