

# ASPI Aspiration Needle

**Description:**

ASPI Aspiration Needle

**Intended use:**

Aspiration needle is used for aspiration of laparoscopic or ultrasound guided transvaginal aspiration.

**Contraindications:**

There are no known contraindications.

**Caution:**

Federal (USA) law restricts this device to sale by or on the order of a physician.

**Warning/ Precaution:**

- Do not use if package is opened or damaged.
- Only experienced surgeon should place this device.
- This device is supplied sterile and intended for single use only.
- Attempt to resterilize a single use device may compromise its structural integrity and/or lead to device failure which may result in patient injury or illness.
- Storage temperature between 5°C to 30°C.
- Dispose in accordance with recognized medical practice and under observance of applicable law and regulation.

**Instruction for Use:**

Using fluoroscopy/ultrasound guidance, advance the needle through anesthetized cut fascia, till it reaches the point of interest.

Further steps should be carried out by the physician as per requirement.














**Do not reuse, reprocess & resterilize:**




Reuse, Reprocessing & reesterilization may compromise the structural integrity, can also create risk of contamination & may not give desired result or create complications, infections which may result in injury, illness or death.

**Limited Express Warranty:**

The limited express warranty as set forth herein is exclusive and in lieu of all warranties of merchantability and fitness for use, remedies, obligations and liabilities for consequential damages.

The products are being sold only for the purpose described herein and such limited express warranty runs only to the original user. In no event shall ALLWIN be liable for any breach of warranty in any amount exceeding the purchase price of the product. ALLWIN reserves the right to make design changes to products without liability to incorporate said changes in ALLWIN products previously designed or sold.

 Catalogue Number	 Batch Code	 Date of Manufacture	 Use By	 Do not re-use	 Do not Re-Sterilize	 Rx only	Caution: Federal Law restricts this device to sale by or on the order of a physician or a practitioner trained in its use.
 Consult Instruction for use	 Do not use if Packing is damaged	 Caution	 Keep out of sunlight	 Keep Dry	 STERILE EO		Sterilized using ethylene oxide

 <b>EC REP</b>	<b>CMC Medical Devices &amp; Drugs S. L.</b> C/Horacio Lengo N° 18, CP 29006, Malaga, Spain Tel.: +34 951 214 054 E-mail : info@cmcmedicaldevices.com	 <b>CE 0123</b>	 <b>Temperature Limit</b>
--	--	--	--

 <p><b>Allwin Medical Devices, Inc.</b> 3305 E. Miraloma Ave., Suite 176 Anaheim, CA 92806 (USA) Tel. : +1 714-572-1709 E-mail : info@allwinmedical.com   www.allwinmedical.com</p>
---